



V.A.C.Ultra™ Therapy System is compatible with a variety of topical solutions for use with a variety of open wounds.

Topical Wound Solutions

The following topical solutions are compatible with V.A.C. VeraFlo™ Therapy.*



Generic Solution Class	Trade Name	Considerations for Use with V.A.C. VeraFlo™ Therapy
Hypochlorite based solutions (e.g. Hypochlorous acid, Sodium hypochlorite)	Dakin's Solution (quarter strength), Dermacyn®, Microcyn®	<ul style="list-style-type: none"> Dakin's Solution should not be used in concentrations greater than 0.125% (quarter strength). Consider using the fewest irrigation cycles and minimizing hold times to the lowest level that is clinically relevant.
Silver nitrate (.5%)	Various	Silver nitrate is light sensitive. Protect V.A.C. VeraLink™ Instillation Tubing from light during use of silver nitrate.
Sulfur-based solutions (sulfonamides)	Mafenide acetate, Sulfamylon®	Refer to manufacturer's labeling for solution-specific considerations. No device related considerations for use with V.A.C. VeraFlo™ Therapy.



*Based on KCI in-house testing of disposables mechanical properties, biocompatibility, and solution interaction. Listing of specific solutions is not an endorsement of or indication of a solution's clinical efficacy. If wound healing goals are not being achieved, consider an alternate instillation frequency, solution concentration, or solution type deemed appropriate by a physician.



For more information on the benefits of V.A.C. VeraFlo™ Therapy, contact your KCI Representative.

NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for KCI products and therapies. Please consult a physician and product instructions for use prior to application. Rx only.

Consult manufacturer's labeling for specific details. V.A.C.Ultra™ Therapy System is indicated for use only with topical wound solutions.

©2011 KCI Licensing, Inc. All rights reserved. Drug trademarks are the property of their respective owners. KCI, Vacuum Assisted Closure® (V.A.C.®), V.A.C.Ultra™, V.A.C. VeraLink™ and V.A.C. VeraFlo™ are trademarks of KCI Licensing, Inc., its affiliates and/or licensors. DSL#11-0018.US • LIT#29-A-212 • Rev. 1/12



Topical Wound Solutions (continued)

The following topical solutions are compatible with V.A.C. VeraFlo™ Therapy.*



Generic Solution Class	Trade Name	Considerations for Use with V.A.C. VeraFlo™ Therapy
Biguanides (Polyhexanide)	Prontosan®	May need to be transferred to a container that can be accessed with a spike.
Cationic solutions (Octenidine, Benzalkonium Chloride)	Octenilin®, Zephiran®	Refer to manufacturer's labeling for solution-specific considerations. No device-related considerations for use with V.A.C. VeraFlo™ Therapy.
Isotonic Solutions	Normal Saline Solution, Lactated Ringer's Solution	Refer to manufacturer's labeling for solution-specific considerations. No device-related considerations for use with V.A.C. VeraFlo™ Therapy.

*Based on KCI in-house testing of disposables mechanical properties, biocompatibility, and solution interaction. Listing of specific solutions is not an endorsement of or indication of a solution's clinical efficacy. If wound healing goals are not being achieved, consider an alternate instillation frequency, solution concentration, or solution type as deemed appropriate by a physician.