



V.A.C.ulta™ Therapy System is compatible with a variety of topical solutions for use with a variety of open wounds.

## Topical Wound Solutions

The following topical solutions are compatible with V.A.C. VeraFlo™ Therapy.\*



Generic Solution Class	Trade Name	Considerations for Use with V.A.C. VeraFlo™ Therapy
<b>Hypochlorite based solutions (e.g. Hypochlorous acid, Sodium hypochlorite)</b>	Dakin's Solution (quarter strength), Dermacyn®, Microcyn®	<ul style="list-style-type: none"> <li>Dakin's Solution should not be used in concentrations greater than 0.125% (quarter strength).</li> <li>Consider using the fewest irrigation cycles and minimizing hold times to the lowest level that is clinically relevant.</li> </ul>
<b>Silver nitrate (.5%)</b>	Various	Silver nitrate is light sensitive. Protect V.A.C. VeraLink™ Instillation Tubing from light during use of silver nitrate.
<b>Sulfur-based solutions (sulfonamides)</b>	Mafenide acetate, Sulfamylon®	Refer to manufacturer's labeling for solution-specific considerations. No device related considerations for use with V.A.C. VeraFlo™ Therapy.



\*Based on KCI in-house testing of disposables mechanical properties, biocompatibility, and solution interaction. Listing of specific solutions is not an endorsement of or indication of a solution's clinical efficacy. If wound healing goals are not being achieved, consider an alternate instillation frequency, solution concentration, or solution type deemed appropriate by a physician.



For more information on the benefits of V.A.C. VeraFlo™ Therapy, contact your KCI Representative.

**NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for KCI products and therapies. Please consult a physician and product instructions for use prior to application. Rx only.**

Consult manufacturer's labeling for specific details. V.A.C.Ultra™ Therapy System is indicated for use only with topical wound solutions.

©2011 KCI Licensing, Inc. All rights reserved. Drug trademarks are the property of their respective owners. KCI, Vacuum Assisted Closure® (V.A.C.®), V.A.C.Ultra™, V.A.C. VeraLink™ and V.A.C. VeraFlo™ are trademarks of KCI Licensing, Inc., its affiliates and/or licensors. DSL#11-0018.US • LIT#29-A-212 • Rev. 1/12



## Topical Wound Solutions (continued)

The following topical solutions are compatible with V.A.C. VeraFlo™ Therapy.\*



Generic Solution Class	Trade Name	Considerations for Use with V.A.C. VeraFlo™ Therapy
<b>Biguanides (Polyhexanide)</b>	Prontosan®	May need to be transferred to a container that can be accessed with a spike.
<b>Cationic solutions (Octenidine, Benzalkonium Chloride)</b>	Octenilin®, Zephiran®	Refer to manufacturer's labeling for solution-specific considerations. No device-related considerations for use with V.A.C. VeraFlo™ Therapy.
<b>Isotonic Solutions</b>	Normal Saline Solution, Lactated Ringer's Solution	Refer to manufacturer's labeling for solution-specific considerations. No device-related considerations for use with V.A.C. VeraFlo™ Therapy.

\*Based on KCI in-house testing of disposables mechanical properties, biocompatibility, and solution interaction. Listing of specific solutions is not an endorsement of or indication of a solution's clinical efficacy. If wound healing goals are not being achieved, consider an alternate instillation frequency, solution concentration, or solution type as deemed appropriate by a physician.